

SteriKleen™



Concentrated, Hospital Grade, Virucidal, Disinfecting, Sanitizing, Fungicidal, Mildewstat Solution DIN # 02500787

Formulated for a wide variety of sanitization & disinfection applications including: Hospitals, Veterinary Clinics, Food Processing Plants & Food Service Prep Areas, Transportation Related Equipment & other Public Contact Settings.

PRODUCT DESCRIPTION:

This product contains no phosphates or phosphorous compounds. SteriKleen is a concentrated non-acidic disinfectant, virucide and cleaner.

Total % Quat in SteriKleen Formulation: 0.344% or 3440 ppm.

SteriKleen can clean, disinfect and deodorize in one labour saving step. SteriKleen can be used with a mop and bucket, trigger sprayer, sponge or by soaking. SteriKleen is effective for use on non-food, and food contact surfaces. Rinsing is required on food-contact surfaces and utensils.

SteriKleen can be used to disinfect hard non-porous surfaces and is rated as effective for destruction of a wide range of viruses and pathogens, among them: Adenovirus Type 5, Hepatitis B & C, HIV 1, Influenza A, RSV, Avian Influenza virus and a wide range of bacterial pathogens, and fungi.

No fragrance added. *Extremely faint 'Baby Powder' scent.*



Effective against (Andenovirus Type 5) and capable of killing SARS-CoV-2 (the virus that causes Covid-19)

VIRUCIDAL DISINFECTION: Use SteriKleen at full strength. Treated surfaces must remain wet for 1 minute

DIRECTIONS FOR USE:

SteriKleen is a proven disinfectant, cleaner, sanitizer, fungicide, mildewstat, virucide which is effective in water up to 400 ppm hardness in the presence of 5% serum contamination.

GENERAL DISINFECTING SURFACES: (1 part SteriKleen to 3 parts clean potable water)

To disinfect hard, non-porous surfaces, spray SteriKleen at 1 part product to 3 parts clean potable water. Allow surfaces to remain wet for 10 minutes. For heavily soiled areas, a preliminary cleaning is required. Prepare fresh solution daily or when use solution becomes visibly dirty.

! NOTE ! This product is not to be used as a sterilant/high-level disinfectant on any surface or instrument that: (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.



Concentrated, Hospital Grade, Virucidal, Disinfecting, Sanitizing, Fungicidal, Mildewstat Solution DIN # 02500787

SANITIZING NON-FOOD CONTACT SURFACES: (1 part SteriKleen to 16 parts clean potable water)

SteriKleen is an effective sanitizer on hard, non-porous surfaces. When used on surfaces such as floors, walls, tables, etc, spray SteriKleen directly on the surface to be sanitized and allow surface to remain wet for a minimum of 60 seconds before cleaning with a clean cloth, paper towel - or rinse with clean potable water.

SANITIZING FOOD CONTACT SURFACES: (1 part SteriKleen to 21 parts clean potable water)

SteriKleen is an effective sanitizer and cleaner for a wide variety of equipment and utensils in food processing plants. Dilute with 21 parts of clean potable water to 1 part of SteriKleen. Rinse surface with clean potable water prior to use. For heavily soiled equipment or utensils, pre-wash with SteriKleen. Apply the solution with a cloth, sprayer device, or immerse utensil or equipment in the solution. Allow surface to remain wet for at least 60 seconds. Allow to air dry then rinse with clean potable water. Prepare a fresh solution for every use. DO NOT reuse pre-used solution for sanitizing.

Wash hands thoroughly with soap and water after handling product. Avoid contact with food.

PRECAUTIONS:

KEEP OUT OF REACH OF CHILDREN. DANGER CORROSIVE TO EYES AND SKIN. DO NOT GET IN EYES, ON SKIN OR ON CLOTHING. HARMFUL IF SWALLOWED OR INHALED. DO NOT SWALLOW. DO NOT INHALE AEROSOLS OR VAPOURS. AVOID CONTAMINATION OF FOOD DURING APPLICATION AND STORAGE OF THIS PRODUCT. WEAR CHEMICAL RESISTANT GLOVES AND GOGGLES WHEN DILUTING THE PRODUCT. WEAR GOGGLES WHEN APPLYING AS SPRAY.

CONTAINS: Linear alcohol ethoxylate, Sodium metasilicate and Tetrasodium EDTA.

FIRST AID:

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician IMMEDIATELY. Remove and wash contaminated clothing before reuse. If swallowed, promptly drink a large quantity of milk, egg whites, gelatin solution, or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician or poison control centre IMMEDIATELY. Take container, label or product name and Drug Identification Number (DIN) with you when seeking medical attention.

TOXICOLOGICAL INFORMATION:

Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

See SteriKleen Technical Data Sheet for more information.

Active Ingredients:

Alkyl (40%C12, 50%C14, 10%C16)	
Dimethyl benzyl ammonium chloride	0.138% w/w
Octyl decyl dimethyl ammonium chloride	0.103% w/w
Dioctyl dimethyl ammonium chloride	0.041% w/w
Didecyl dimethyl ammonium chloride	0.062% w/w
Other Ingredients	99.943% w/w
Total:	100% w/w

Proudly made in Canada by



INTECH Environmental Canada Corp
Surrey BC Canada, V3V3K2, 1.877.658.3682